SANZEGF

SPORT & MEDICAL STOCKINGS

ACTIVE

IT	ISTRUZIONI PER L'USO
EN	INSTRUCTIONS FOR USE
LT	NAUDOJIMO INSTRUKCIJOS
PT	INSTRUÇÕES DE UTILIZAÇÃO
DE	GEBRAUCHSANWEISUNGEN
FR	MODE D'UTILISATION
HU	HASZNÁLATI ÚTMUTATÓ
SK	NÁVOD NA POUŽÍVANIE
KO	사용 방법
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

SANZEG

SPORT & MEDICAL STOCKINGS

SPORT & FITNESS

ITALIANO	4
ENGLISH	6
LIETUVIŲ	8
PORTUGUÊS	10
DEUTSCH	12
FRANÇAIS	14
MAGYAR	16
SLOVENSKÝ	18
한국어	20
ΕΛΛΗΝΙΚΗ	22

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Athletic performance begins with making the right choices.

Dear Customer, Thank you for choosing a Sanyleg Compression Sport garment. The experience we have gained in the field of medical stockings has allowed us to develop a line of athletic socks with a high degree of technical content and maximum attention to design detail. The result is a highly breathable garment that offers superior comfort.

The heel section provides for:

- the proper placement of the heel in the heel pocket, thus providing for greater stability.
- proper positioning at the maximum compression point, as well as a better fit and greater wearability and elasticity around the circumference of the ankle.

Directions

Sanyleg Active socks are ideal for sports activities; our garments are manufactured using materials of the highest quality, with an exceptional technical profile. The pleasant feeling of lightness and wellness is obtained thanks to the dynamic effect of the product's graduated compression, which decreases from the ankle upward. This helps to increase blood circulation in the legs, thus improving muscle oxygenation.

General contraindications

Advanced state peripheral arterial occlusive disease - Massive oedema of the lower limbs of a pathological nature - Septic phlebitis - Severe phlebitis, such as phlegmasia cerulean dolens - Lymphangitis - Advanced peripheral neuropathies (i.e., diabetes mellitus) - Other sensory disorders of the limbs - Intolerances/allergies related to the material of the compressive stockings.

Relative contraindications

Ulcers and open wounds - Moist dermatosis - Immobility - Dermatitis stasis - Valvular incontinence of the superficial and deep venous system. - Deep vein thrombosis or acute thrombophlebitis - Post thrombophlebitic syndrome - For stockings with silicone edges, skin irritation may arise in cases of hypersensitivity to the material itself.

Composition of the materials

The textile composition of each individual product is indicated on the outside of the package. The materials with which our products are manufactured are listed hereafter.

CO: Cotton **EA:** Elastane **PA:** Polyamide

WO: Wool **PP:** Polypropylene **SE:** Silk

Instructions and recommendations for wearing Sanyleg sock

Protect the socks against contact with any oils, creams, and ointments that may be applied to the legs. Rough fingernails or rings with rough edges could potentially damage the product's fabric. If the skin is wet,

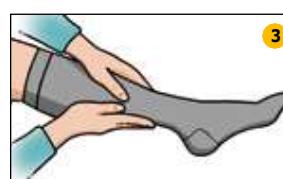
apply talcum powder to the foot in order to facilitate its insertion into the sock.

1. Turn the sock inside-out up to the heel - Use both hands to widen the foot portion and slide the foot into the sock up to the heel.

2. Turn back the inside-out portion of the sock, taking care to position the heel properly.

3. Pull the rest of the sock up to the knee, smoothing out the fabric with the palm of your hand. Never pull directly on the top edge of the sock.

Washing instructions

Sanyleg stockings can be washed by hand or in the washing machine.

- **Hand washing:** use detergents for delicate fabrics. Rinse thoroughly. Do not wring. Air dry.
- **Washing machine:** place the garment in an appropriate bag or in a pillow case. Do not bleach. Do not machine dry.
- **Do not dry clean. Do not iron.**

Warning

The manufacturer shall bear no responsibility for any personal injuries and/or property damage that may occur in the event that the product is used improperly or for purposes other than those for which it is intended.

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Sanyleg S.r.l. Unipersonale