(Fig. 1)

(Fig. 2)

(Fig. 3)

(Fig. 4)

(Fig. 5)

## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

### INTENDED USE/INDICATIONS

The DonJoy Airstrap is designed to provide comfort and support of the lumbar spine. It may be suitable for the symptomatic relief of acute and chronic low back pain.

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

### CONTRAINDICATIONS

If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. Do not use if you are allergic to any of the materials contained within this product.

### WARNINGS AND PRECAUTIONS:

- For maximum efficacy, ensure the correct size of strap is selected and the side pulls are adjusted to the required fit.
  - Do not use over open wounds.
  - Do not use the device if it is damaged and/or the packaging has been opened.
  - If you experience any pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS

1 - Before positioning the back support, inflate it slightly with the removable hand pump. Place the back support around the waist, taking care to centre it on either side of the spine. Position the back support according to the area of the spine requiring support. The inflation valve should be located on the patient's right-hand side.

2 - Inflate the back support using the removable hand pump. (Fig. 1 & 2)

A - To insert and remove the hand pump: Hold the inflation valve on the back support, insert or remove the hand pump by giving it ¼ rotation movement. (Fig. 3)

B - To inflate the back support: Insert the transparent tip of the hand pump into the valve (push it in quite deep to allow air flow). Inflate the back support to the required level of comfort.

C - To deflate the back support: Reverse the position of the hand pump and insert the black tip of the hand pump into the valve (push it in quite deep to release air).

3 - When the back support is inflated, pull and adjust the elastic side straps according to the level of comfort required. Loosen the elastic side straps before removing the back support. (Fig. 4 & 5)

### CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

### COMPOSITION

Polyamide, Polyester, PVC, Rubber, Elasthan

 MADE WITH NATURAL RUBBER LATEX.

### INTENDED FOR USE ON A SINGLE PATIENT

**WARNING:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

**NOTICE:** WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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© 2020 DJO, LLC