
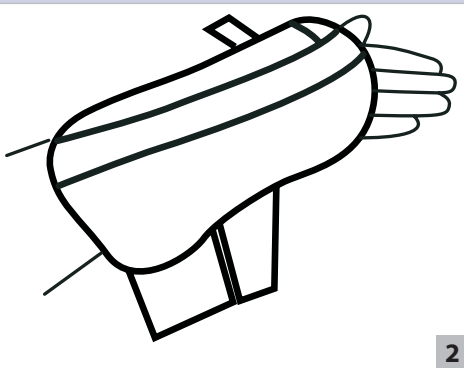
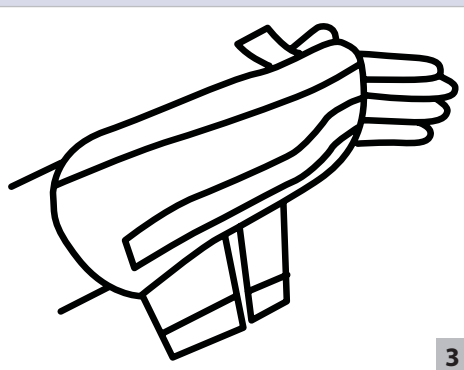
1

2

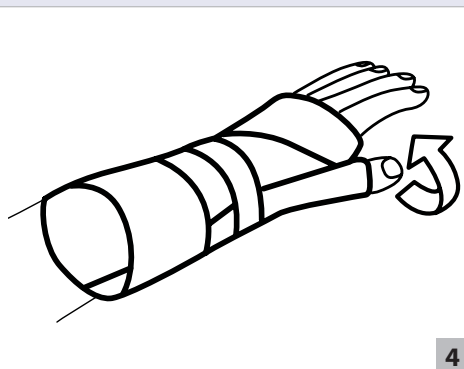3

4

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

- Ideal for sprains, arthritis, joint and ligament injuries to the wrist and thumb.

PERFORMANCE CHARACTERISTICS

- Providing immobilization or controlled movement of the limb or body segment.

CONTRAINDICATIONS

- None

WARNINGS AND PRECAUTIONS

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Do not use over open wounds.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

- 1) Fit the brace around the injured wrist (left or right). Place the thumb in the casing at the top of brace.
- 2) Shape and place the palmar stay on the inside of the hand and wrist.
- 3) Shape and place the aluminum thumb stay to block the thumb in abduction.
- 4) Tighten and secure contact closure straps.
- 5) Then tighten and secure contact closure thumb strap around the thumb.
- 6) Adjust the tightness of the straps according to compression and comfort needed.

CARE

Hand-wash in cold water with mild soap and air-dry. Do not bleach. Do not dry-clean or machine-dry. Do not iron.

NOT MADE WITH NATURAL RUBBER LATEX.

FOR SINGLE PATIENT USE ONLY

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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