Loop	REF	UDI-DI
	01-100001	05420063004084
	01-100002	05420063000007
	01-100003	05420063000024
	01-100004	05420063000031
	01-100005	05420063000048
01-100006	05420063004091	
01-100011	05420063004107	

UK REP	MVS in Motion	Qarad UK Ltd.
	Krommebeekpark 11	8 Northumberland Ave
UK CE MD	8800 Roeselare (BE)	Westminster, London WC2N 5BY
	mvs-in-motion.com	United Kingdom

EN – MEDICAL DEVICE – INSTRUCTIONS FOR USE

Intended purpose: To tone, shape, strengthen or rehabilitate muscle groups through resistance exercise, for use in physical therapy or fitness training. **Intended user:** Lay person or healthcare professional.

Indications: Medical conditions resulting in limitation in movements, postural impairments and muscle performance impairments. **Contra-indications:** Use should be discontinued in case of unusual swelling, discomfort, dizziness, severe muscle or joint pain or pain in the chest.

Precautions:
 - Consult a healthcare professional before starting an exercise program;
 - Always examine MoVeS (FIT) Loop / Wide Loop / Superloop before use for cuts, small tears or punctures. Do not use if damaged;
 - Store away from sharp objects. Remove rings and be aware of sharp fingernails;
 - Store away from sunlight.

Warnings:
 - This device contains natural rubber latex which may cause allergic reactions;
 - MoVeS (FIT) Loop / Wide Loop: Do not stretch beyond 150cm;
 - MoVeS FIT Superloop: Do not stretch beyond 6m.

Report any serious incident (death, serious deterioration of health, serious public health threat) to the Manufacturer and your Competent Authority.

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