ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

The cervical collar DonJoy C2+™ is designed to stabilise and relieve the cervical spine and may be suitable for the symptomatic relief of pain and stiffness in the cervical spine area associated with conditions such as strains, sprains, stiff neck and whiplash injuries.

Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction. Providing immobilization or controlled movement of the limbo or body segment.

CONTRAINDICATIONS

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- Any known allergies to the medical device materials.

WARNING & PRECAUTIONS

The product is intended for single patient use only. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact a medical professional immediately. Note: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

- 1) Undo the hook and loop closure to open the collar.
- 2) Place the collar around the neck, with the anatomically contoured part under the chin and the hook and loop closure on the back.
- 3) Close the collar by using the hook and loop closure. Improvement of immobilization and comfort is achieved by minor adjustments of the strap.

A) Adjustable support
If needed, the rigid insert can be removed for a less rigid support. Open the slit at the level of the strap and remove the stiffener located in between the two foams. The rigid insert can easily be put back into place for additional support needs.

B) Removable covers
The DonJoy C2+™ cervical collar is supplied with 2 covers. If needed, replace the cover with the other one.

CLEANING INSTRUCTIONS

Cervical collar: hand wash (max 30°C) using mild soap.
Removable covers: machine washable (max 40°C) with mild detergent. Hook and loop should be closed for washing.
Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.

COMPOSITION

Bamboo, Cotton, Polyamide, Polyurethane, Polyester, Polypropylene, Silicone

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

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