# Stabilax

## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

### INDICATIONS:

The DonJoy Stabilax is designed to support the knee. It may be suitable for use following mild medial and lateral ligament injury or meniscal injury and chronic knee instability. It may also help to enhance proprioception. Soft-good/semi-rigid designed to restrict motion through elastic or semi rigid construction.

### CONTRAINDICATIONS/PRECAUTIONS:

Do not use if you are allergic to any of the materials contained in this product.

### WARNINGS AND PRECAUTIONS:

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
Do not use over open wounds.
Do not use this device if it was damaged and/or package has been opened.
NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### APPLICATION INSTRUCTIONS:

- Grasp edges of sleeve at the larger opening. Pull the sleeve onto the knee.
Position the sleeve so that the hinges are parallel to each other.
During application, if the hinges come out of their pockets, re-position them (Fig. 1-6)
Pull the support over the knee, so that the circular buttress covers the kneecap evenly.
Hinges should be parallel and positioned on the inside and outside of the knee.
Fasten the support straps at the thigh and calf.

Removable polycentric hinges (Fig. 1-6)

NB - Do not bend the kneesleeve perpendicular to the axis of rotation of the hinges.

### CARE INSTRUCTIONS:

Hand wash in cold water with mild soap, rinse thoroughly. Air dry. Do not machine dry or bleach.

### COMPOSITION:

Polyamide, Spandex, Nylon, Polyester, Jersey, Cotton, Polyurethane

FOR SINGLE PATIENT USE ONLY. NOT MADE WITH NATURAL RUBBER LATEX.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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