GYMNIC®

The Way to Move

CE

SAFE PRODUCT

- LATEX FREE ✓
- PHTHALATE FREE ✓
- BPA FREE ✓
- LEAD FREE ✓
- REACH COMPLIANT ✓

WARNING!
For body weight
exercise only.

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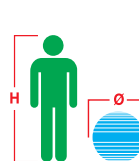
STANDARD QUALITY: GYMNIC CLASSIC - PHYSIO GYMNIC - OPTI BALL ACTIVITY BALL - JINGLIN' BALL - THERASENSORY GYM BALL - SIT'N'GYM JR.


- EN In case of puncture the ball bursts.
- FR En cas de crevaison le ballon éclate.
- IT In caso di foratura la palla scoppia.
- DE Bei Löchern platzt der Ball.
- ES En caso de pinchazo, la bola se revienta.
- BG В случай на пробиване топката се пръска (гръмна).
- CS V případě propíchnutí míč praskne.
- DA Hvis bolden punkterer, eksploderer den.
- EL Αν τρυπήσει η μπάλα σκάει.
- ET Katki torgatud pall puruneb.
- FI Pallo räjähtää puhjetessaan.
- HR U slučaju oštećenja dolazi do pucanja lopte.
- HU Ha a labda kilyukad, kipukkad.
- LT Prakiurus k kamuolių sprogtą.
- LV Cauruma gadījumā bumba uzsprāgst.
- NL Bij doorboring ontploft de bal.
- NO Hvis ballen punkteres, vil den sprekke.
- PL Em caso de furo a bola estoura.
- PT Em caso de furo a bola estoura.
- RO In caz de perforare mingea explodează.
- RU В случае прокола мяч лопнет.
- SK V prípade prepichnutia lopty praskne.
- SL V primeru predrtja žoga poči.
- SV I händelse av punktering kommer bollen att spricka.
- TR Delinme halinde top patlar.
- UK У разі проколу шар розривається.
- HI चंवर होने पर गैद (बॉल) फट जाती है।
- JA バンクの際、ボールは破裂します。
- KO 구멍이 나면 공은 터집니다.
- ZH 遇到刺穿，球体爆裂。

Height range H		Ball size Ø	
m	ft	cm	in
< 1,35	< 4'4"	30-35	12"-14"
1,35-1,54	4'4"-5'0"	45	18"
1,55-1,69	5'1"-5'5"	55	22"
1,70-1,84	5'6"-6'0"	65	26"
1,85-2,04	6'1"-6'7"	75	30"

GROUP ACTIVITY	Ball size Ø	
	cm	in
GROUP ACTIVITY	85	34"
	95	38"
	120	48"

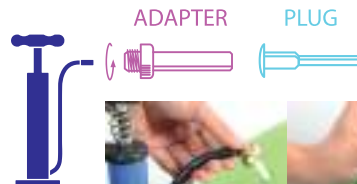**RECOMMENDED WEIGHT
MAX 120 KG (270 LBS)**
**TESTED LOAD CAPACITY
MAX 300 KG (660 LBS)**

WARNING: FOR BODY WEIGHT EXERCISE ONLY. NOT SUITABLE FOR EXERCISES WITH BARBELL OR ANY KIND OF WEIGHT. LIFTING WEIGHTS WHILE USING THE PRODUCT MAY CAUSE INJURIES TO THE USER.

INFLATION. IT MUST BE PERFORMED BY AN ADULT. THE DIAMETER SHOWN ON THE PRODUCT MUST NOT BE EXCEEDED.

- Please remove the plug from the product and proceed with inflation.
- Inflate with a pump using the special adapter (Photo 1) if necessary. MAX ATM 0.1 - PSI 1.5.
- Inflate at room temperature (20° C / 70° F) in places free from any pointed objects.
- In order to reach the indicated diameter an energetic inflation at intervals is required.

CAUTION. IN CASE IT IS USED BY CHILDREN, THE ADULT SUPERVISION IS REQUIRED.

- The product integrity must be checked before every use. Use it away from direct heat and on surfaces free from pointed objects (pay attention to zippers, rings, watches, etc.).
- Clean with water and neutral soap. Do not use any brushes, abrasive sponges or chemical substances. Do not use sterilization systems.
- In case of puncture DO NOT REPAIR. The repaired product does no longer comply with the safety requirements.
- Do not expose the product to direct heat or to direct light for a long time. This causes the alteration of its physical and chromatic features.

THE MANUFACTURER AND THE DISTRIBUTOR ARE NOT RESPONSIBLE FOR ANY DAMAGE CAUSED BY AN INCORRECT USE OF THE PRODUCT. READ THIS INSTRUCTION MANUAL CAREFULLY AND KEEP IT FOR ANY FUTURE USE.

In case of claim mention the TRACKING NUMBER - LOT shown on the product.

MANUFACTURED BY: Ledraplastic S.p.A. - Via Brigata Re, 1 - 33010 Osoppo (UD) - ITALY - www.gymnic.com

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