(Fig. 1)

(Fig. 2)

(Fig. 3)

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

The DonJoy Duostrap is designed to provide comfort and support of the lumbar spine and abdomen. It may be suitable for the symptomatic relief of acute and chronic low back pain. The brace is also indicated for those requiring abdominal support due to an abdominal hernia or muscle weakness. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

CONTRAINDICATIONS

- Pregnancy.
- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional. Do not use if you are allergic to any of the materials contained within this product.

WARNING AND PRECAUTIONS

- As this belt provides abdominal support, please consult your medical professional if using post-operatively as there is an increased risk of blood clots.
 - For maximum efficacy, ensure the correct size of strap is selected and the side pulls are adjusted to the required fit.
 - If you experience any pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your medical professional immediately.
 - Do not use over open wounds.
 - Do not use the device if it is damaged and/or the packaging has been opened.
- NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

1. Release the lateral elastic side pulls. (Fig.1)
2. Position the belt around the waistline, placing it over the sacro-lumbar area of the patient. Close the belt using contact closure. (Fig.2)
3. Pull the elastic side pulls to the desired level of support and affix them on the front of the belt, using contact closure. (Fig.3)

CARE

Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION

Polyamide, Polyester, PVC, Rubber, Elastane, Steel

 MADE WITH NATURAL RUBBER LATEX.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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